



PARAGON

DISPOSABLE SINGLE USE GARMENT

Instructions for Use

1. Check cooler to ensure proper amounts of water and ice.
2. Put the vest or poncho on, with the "V" neck in the front, and adjust fit with Velcro. The garment may be worn over surgical scrubs and under sterile gowns or lead aprons.
3. Connect the male connectors from the garment into the female connectors on the water supply hose or the cooling system. An audible click can be heard when the connectors are properly connected.
4. Power the system on.
5. Discontinue use if garment becomes uncomfortable, starts to cause excessive sweating, chills, shivering or skin irritation.

Not intended for patient use

1146 Carding Machine Road

Loudon, Tennessee 37774

865.643.8700

www.paragonproinc.com



PARAGON

DISPOSABLE SINGLE USE GARMENT

Instructions for Use

1. Check cooler to ensure proper amounts of water and ice.
2. Put the vest or poncho on, with the "V" neck in the front, and adjust fit with Velcro. The garment may be worn over surgical scrubs and under sterile gowns or lead aprons.
3. Connect the male connectors from the garment into the female connectors on the water supply hose or the cooling system. An audible click can be heard when the connectors are properly connected.
4. Power the system on.
5. Discontinue use if garment becomes uncomfortable, starts to cause excessive sweating, chills, shivering or skin irritation.

Not intended for patient use

1146 Carding Machine Road

Loudon, Tennessee 37774

865.643.8700

www.paragonproinc.com